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EN	INSTRUCTIONS FOR USE - Cinesteam® - Secondary anti-odour absorbent dressing
<p>DESCRIPTION</p> <p>Cinesteam® is a non-adhesive secondary dressing designed to absorb exudates and eliminate unpleasant odours. It consists of an absorbent layer together with a sachet containing cinnamon. Composition: polyethylene, polyester, polypropylene and cinnamon.</p> <p>Cinnamon adsorbs unpleasant-smelling volatile compounds emanating from the wound and masks residual odours with spice natural fragrance.</p> <p>INDICATIONS</p> <p>Cinesteam® is a secondary dressing indicated for the management of malodorous wounds.</p> <p>CONTRAINDICATIONS</p> <p>Cinesteam® should not be used on patients with known sensitivity to any of its components, in particular cinnamon.</p> <p>DIRECTIONS</p> <p>Cinesteam® should be applied on top of an appropriate primary dressing (it is not intended to be in direct contact with the wound).</p>	<p>The absorbent layer (white face) should be applied facing the wound, over the primary dressing.</p> <p>Application of Cinesteam®</p> <ol style="list-style-type: none"> Select the size of Cinesteam® that covers the primary dressing. If necessary, more than one dressing Cinesteam® may be used. Remove Cinesteam® from its primary packaging and apply it directly onto the primary dressing. Keep Cinesteam® in place with any suitable medical material <p>Changing Cinesteam®</p> <p>Gently peel off Cinesteam®.</p> <p>Cinesteam® can be left in place as long as its absorption and odour masking capacities are satisfactory (1 to 7 days). Frequency of change should be dependent on the level of exudate.</p> <p>PRECAUTIONS</p> <p>Do not use Cinesteam® as a primary dressing. Do not cut Cinesteam® because this could cause cinnamon leakage.</p>

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Cinesteam® should not be used if its primary packaging is open or damaged or if the dressing itself is torn.

Do not use Cinesteam® together with an active charcoal dressing, which may impair its odour masking ability.

Do not cover Cinesteam® with an occlusive fastening material or dressing.

Do not eat the cinnamon of Cinesteam®.

Do not bathe or shower when wearing Cinesteam®.

Do not immerse Cinesteam® in water.

Keep out of the reach of children.

Report any serious incidents which occur in connection with Cinesteam® to Cemag Care and to the healthcare competent authority of your country.

If you have any further questions, ask your doctor, pharmacist or nurse.

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